
AMBULANCE SECTOR LEVEL 3 PPE POWERED RESPIRATOR HOOD - DONNING

DONNING

LEVEL 3 PPE

POWERED RESPIRATOR

HOOD

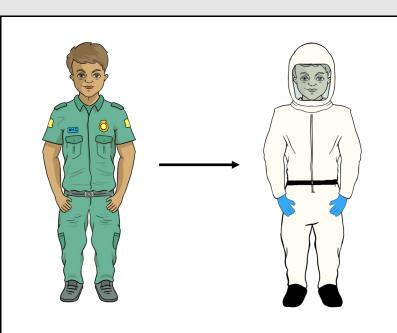


DONNING ORDER

PERFORM HAND HYGIENE

Step 1 - Don fluid repellent coverall


- · Step into coveralls
- Pull up over waist
- Insert arms into sleeves, if thumb hoops available then hoop these over your thumbs.
- Pull up over the shoulders
- Fasten zip all the way to the top

It is not necessary to apply the hood of the coverall as there is no requirement for airborne transmission. This should be tucked into the back of the coverall.

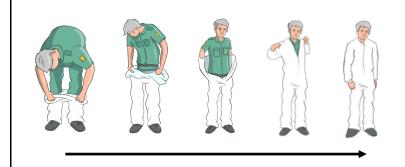

Step 2 - Don powered respirator hood

- Perform safety checks as per guidance sheet
- Apply the belt-mounted respirator unit to the waist and buckle securely and comfortably. Ensure you can easily move the unit to your side to assist with the next steps
- Attach the breathing tube and switch on the powered respirator ensuring a solid green light appears after the testing cycle
- Put on the respirator hood
- Ensure the respirator hood is comfortable and secure

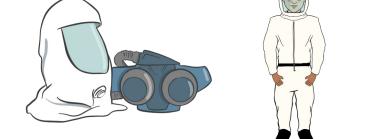


Step 3 - Don disposable gloves

- Perform Hand Hygiene
- · Select according to size
- Apply gloves and extend to wrist, ensure cuff of coverall is covered



AMBULANCE SECTOR LEVEL 3 PPE POWERED RESPIRATOR HOOD - DOFFING

DOFFING LEVEL 3 PPE POWERED RESPIRATOR HOOD

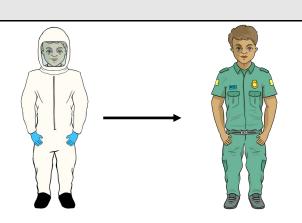

DOFFING ORDER

Step 1 - Remove disposable gloves

- Grasp the outside of the glove with the opposite gloves hand and peel off
- Hold the removed glove in the gloved hand
- Slide the fingers at the un-gloved hand under the remaining glove at the wrist
- Peel the second glove off over the first glove and discard





PERFORM HAND HYGIENE

Step 2 - Remove respirator hood

- Reach to the rear of the head to firmly grip the hood, reach under the chin to find the tag at the bottom of the hood
- Use both hands to move the hood in a forward and downward motion away from the face/ head
- · Unbuckle the waist belt
- · Switch off the respirator






PERFORM HAND HYGIENE

Step 3 - Remove fluid repellent coverall

- Tilt the head back and with one hand pull the coveralls down and away from your body
- With the other hand, run your hand up the zip until you reach the top and unzip the coveralls completely without touching any skin or uniform following the guidance of your crewmate
- Remove coveralls from top to bottom. After freeing shoulders, pull arms out of the sleeves
- Roll the coverall, from the waist down and from the inside of the coverall, down to the top of the boots, taking care to only touch the inside of the coveralls
- Use one boot to pull off the coverall from the other boot and vice versa then step away from the coverall ensuring the coverall has been removed and dispose of it as clinical waste.
- NB: It may be beneficial to cut small slits in the bottom of the legs of the coveralls to allow for easier removal



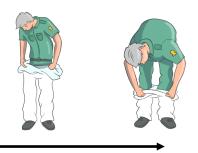

PERFORM HAND HYGIENE